
<b>Baxter</b>				Artwork-Ersteller: Artwork creator:	Techn. Kontrolle: Techn. control:	Imprimatur QO B. Gaisböck 13.01.2010 (LCR688/09)
<b>Packaging Design &amp; Development</b> Date: see page header						
Materialnummer: Material number:	6208400ZA07	Versionsnummer: Update version:	2	Größe: Size:	420 x 594	
Farben: Colors:	pms280c		Techn. Spez.: Techn. Spec.:		TSBP26	

6208400ZA07

## TISSEEL/ARTISS Spray Set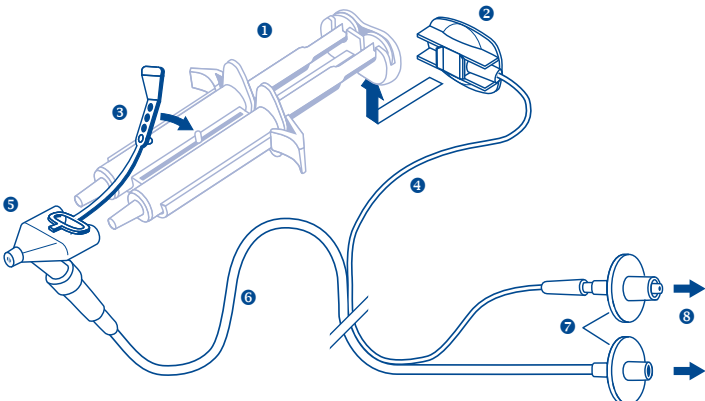

ENG 1 Double Syringe System 2 Clip 3 Pull Strap 4 Sensor Line 5 Spray Head 6 Pressure Line 7 Sterile Filters 8 EASYSPRAY Pressure Regulator	FRE 1 Système à double seringue 2 Support 3 Langnette de sécurité 4 Tubulure de détection 5 embout-pulvérisateur 6 Tubulure sous pression 7 Filtres stériles 8 Dispositif de contrôle de la pression EASYSPRAY	NOR 1 Dobbeltspøytesystem 2 Klemme 3 Stropp 4 Sensorslange 5 Sprayhode 6 Trykkslange 7 Sterile filtre 8 EASYSPRAY-trykkregulator
BUL 1 Система с двукамерна спринцовка 2 Щипка 3 Издърпваща се лента 4 Сензорна линия 5 Пулверизираща глава 6 Линия за налягане 7 Стерилни филтър 8 Регулатор на налягане EASYSPRAY	GER 1 Doppelspritzensystem 2 Clip 3 Sicherungslasche 4 Sensorleitung 5 Sprühkopf 6 Druckleitung 7 Sterilfilter 8 EASYSPRAY Druckregler	POL 1 system dwustrzykawkowy 2 klamra 3 taśma zabezpieczająca 4 linia czujnikowa 5 głowica rozpylająca 6 linia ciśnieniowa 7 sterylne filtry 8 regulator ciśnienia EASYSPRAY
CZE 1 systém dvojité stříkačky 2 svorka 3 zatahovací pásek 4 snímací linka 5 sprejovací hlavičce 6 tlaková linka 7 sterilní filtry 8 regulátor tlaku EASYSPRAY	GRC 1 Σύστημα διπλής σύριγγας 2 Σύνδεσμος 3 Τακτρία έκλξης 4 Γραμμή αισθητήρα 5 Κεφαλή ψεκασμού 6 Γραμμή πίεσης 7 Στείρα φίλτρα 8 Συσκευή ρύθμισης πίεσης EASYSPRAY	POR 1 Sistema de dupla seringa 2 Clip 3 Tira de fixação 4 Tubo sensor 5 Ponta de pulverização 6 Tubo de pressão 7 Filtros estériles 8 Regulador de pressão EASYSPRAY
DAN 1 Dobbeltkammersprøjte 2 Klips 3 Strop 4 Sensorslange 5 Spraydyse 6 Trykkslange 7 Sterile filtre 8 EASYSPRAY trykregulator	HRV 1 Sustav s dvije štrcaljke 2 Stezaljka 3 Stezni remen 4 Vod senzora 5 Glava raspršivača 6 Vod za tlak 7 Sterilni filtri 8 Regulator tlaka EASYSPRAY	SPA 1 Sistema de jeringa doble 2 Pinza 3 Banda de anclaje 4 Línea sensora 5 Cabezal pulverizador 6 Línea de presión 7 Filtros estériles 8 Regulador de presión EASYSPRAY
DUT 1 Dubbel spuitstelsysteem 2 Houder 3 Veiligheidsclip 4 Sensorslang 5 Spraytip 6 Drukslang 7 Sterilisatiefilters 8 EASYSPRAY-drukregelaar	HUN 1 Kettős fecskendőrendszer 2 Kapocs 3 Rögzítő heveder 4 Erzékelő vonal 5 szűrőfej 6 Nyomás vonal 7 Steril szűrők 8 EASYSPRAY nyomásszabályozó	SWE 1 Dubbelt sprutsystem 2 Hållare 3 Säkerhetsspärr 4 Sensorslang 5 Sprayhuvud 6 Tryckslang 7 Sterila filter 8 EASYSPRAY tryckregulator
FIN 1 Kaksoisruiskupidike 2 Pidike 3 Lukituspidike 4 Anturiletetu 5 Sumutinkärki 6 Paineletetu 7 Steriilit suodattimet 8 EASYSPRAY-paineensäätölaite	ISL 1 Tvöföld sprauta 2 Klemma 3 Togband 4 Skynjaraleiðsla 5 Úðahaus 6 Þrýstislanga 7 Sótthreinsisúrir 8 EASYSPRAY þrýstjafnari	TUR 1 Çift Enjektör Sistemi 2 Klips 3 Çekme Şeridi 4 Sensör Hattı 5 Püskürtme Kafası 6 Basing Hattı 7 Steril Filtreler 8 EASYSPRAY Basing Regülatörü
ITA 1 Sistema a doppia siringa 2 Clip 3 Linguetta 4 Condotto del sensore 5 Nebulizzatore 6 Condotto di erogazione 7 Filtri sterili 8 EASYSPRAY – apparecchio regolatore di pressione		

### INSTRUCTIONS FOR USE

ENG

## TISSEEL/ARTISS Spray Set

For the Simultaneous Application of TISSEEL or ARTISS Two-Component Fibrin Sealant. The TISSEEL/ARTISS Spray Set is used in conjunction with a double syringe system and the EASYSPRAY Pressure Regulator.

### 1. FIELD OF APPLICATION

The double syringe systems and TISSEEL/ARTISS Spray Set have been designed for the application of TISSEEL or ARTISS fibrin sealant onto broad wound areas. It has been found especially useful for indications where the application of a uniform fibrin sealant film is essential.

### 2. DESCRIPTION

Spray set consisting of:

1 Spray Head

1 dual-lumen connection tube consisting of a pressure line with a transparent sterile filter, and a thinner sensing line with a blue sterile filter and a clip to be attached to the double syringe system plunger for gas activation.

#### 2.1 Principle

Inside the spray head, which is attached to the double syringe system containing the TISSEEL or ARTISS Sealer Protein Solution and Thrombin Solution, the two solutions are passed through two adjacent outlets. These two openings are encircled by the pressurized gas exit. The gas flow atomizes and mixes the two solutions. The pressure and flow rate of the pressurized gas are controlled by the EASYSPRAY Pressure Regulator. The spray set must be used only in connection with the EASYSPRAY Pressure Regulator. The gas flow is activated by occluding the opening on the clip attached to the double syringe system plunger with the thumb.

### 2.2 Symbols and explanations

	Caution
	Single use
	Do not use if the product, its sterile barrier system or its packaging is damaged or shows any sign of deterioration
	Latex free
	Sterilized by ethylene oxide
	Temperature limitation (store between 2°C to 30°C or between 36°F to 86°F respectively)
	Manufacturer
	Batch number
	Use by
	Catalogue number
	Consult instructions for use
	Contains or presence of phthalate: bis(2-ethylhexyl) phthalate (DEHP)
	Keep dry

Non-pyrogenic in unopened and undamaged package.

Phthalates:

Only tubings contain DEHP. No DEHP extracted by pressurized gas.

### 3. OPERATING INSTRUCTIONS

**3.1** Prepare the double syringe system containing the TISSEEL or ARTISS Sealer Protein Solution and the Thrombin Solution.

**NOTE:** Remove surplus air from the syringes.

**3.2** Attach the spray head firmly to the cones of the double syringe system. Fasten the pull strap to the double syringe system to assure that the spray head is secured tightly. Should the pull strap tear, further use is still possible after careful examination of the connection site. In such a case, however, there is a potential risk of fluid leakage.

**3.3** Fit the pressure line to the lower luer-lock connector of the spray head. Connect the transparent sterile filter at the other end of the tube to the appropriate connector on the EASYSPRAY Pressure Regulator. Connect the EASYSPRAY Pressure Regulator to the gas source according to the corresponding IFU

**3.4** Attach the clip of the sensor line by sliding it into the grooves on the back end of the double syringe system plunger. The clip is locked onto the plunger when an audible "click" is heard.

**3.5** Connect the blue sterile filter at the other end of the line to the appropriate luer-lock connector (blue ring) on the EASYSPRAY Pressure Regulator.

**3.6** To activate the flow of gas, occlude the opening in the clip center with thumb. A pressure of 1.5 – 2 bar (21.8 – 29 psi) has proved suitable. With that 5 – 20 litres of sterile gas are released per minute.

**3.7** The stream of sterile gas ejected from the spray head can be used to displace the tissue fluid on the operation area, if necessary, and thus create optimum conditions for the application of TISSEEL or ARTISS Fibrin Sealant.

**3.8** The quantity of TISSEEL or ARTISS Fibrin Sealant is controlled by depressing the double syringe system plunger.

**3.9** The EASYSPRAY Pressure Regulator will continue to emit gas for a brief period after the thumb is removed from the clip/plunger. This delay helps to avoid clogging of the spray head.

**3.10** To achieve an optimal result, a spraying distance of 10 – 15 cm is recommended. In specific situations where spraying at less than 10 cm is necessary, the gas pressure must be reduced appropriately.

**3.11** TISSEEL or ARTISS Fibrin Sealant, containing 1 ml of Sealer Protein solution and 1 ml of Thrombin Solution is sufficient for spraying an area of up to 100 cm<sup>2</sup>, depending on the desired film thickness.

### 4. WARNING

Any application of pressurized gas may be associated with a potential risk of air embolism, tissue rupture or gas entrapment with compression, which may be life-threatening. Especially in case of low central venous pressure there is an increased risk of air embolism in vena cava anastomoses. Be sure to take appropriate measures to exclude these risks, by control of maximum pressure (2 bar / 29 psi) and minimum distance (10 cm).

The spray set must only be used in connection with the EASYSPRAY Pressure Regulator. DO NOT connect the spray set directly to the pressurized gas supply. The use of accessories of other manufacturers is not permitted.

Do not re-sterilize

Reuse of single-use devices creates a potential risk of patient or user infections. Contamination of the device may lead to injury, illness or death of the patient.

### 5. ACCESSORIES

EASYSPRAY Pressure Regulator

BAXTER, TISSEEL, ARTISS and EASYSPRAY are trademarks of Baxter International Inc., its subsidiaries or affiliates.

Baxter AG, 1221 Vienna, Austria

**USA** Distributed in the USA by:  
Baxter Healthcare Corporation  
Deerfield, IL 60015 USA  
1-800-423-2090  
Rx only

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Baxter AG, 1221 Viyana, Avusturya